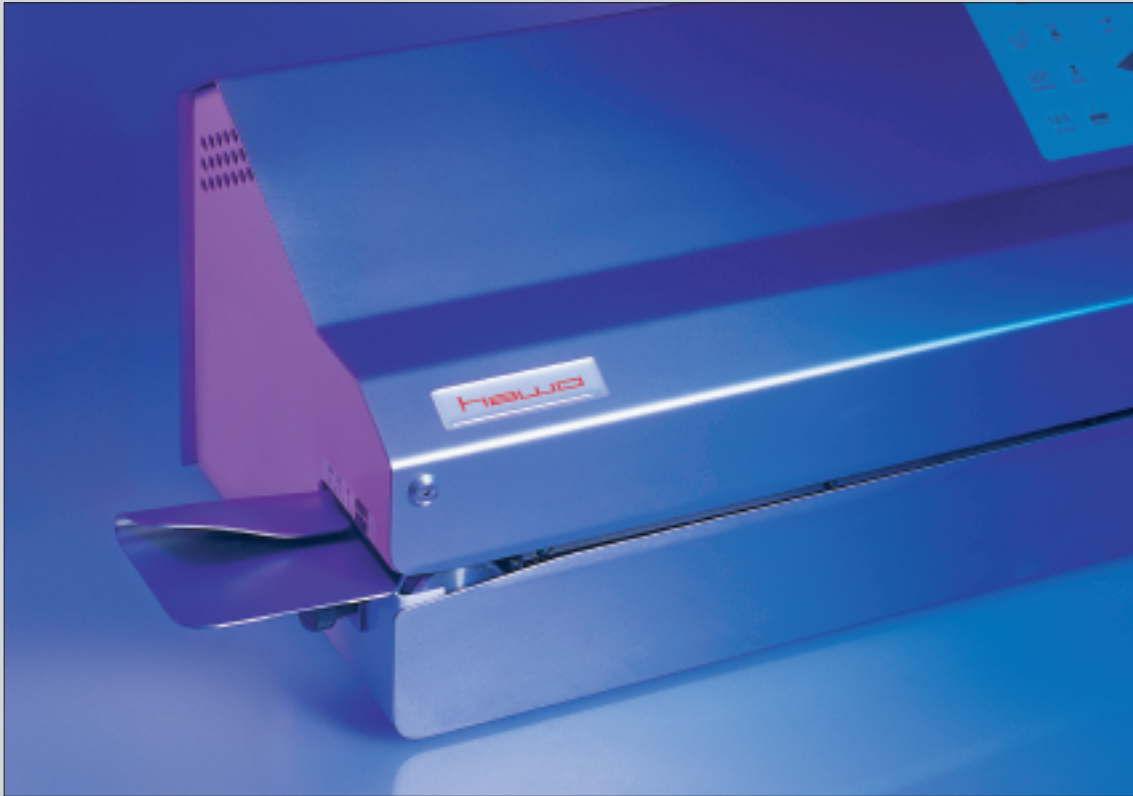
hm 800 DC

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Rotary sealer with microprocessor-controlled printing unit

corresponding to DIN 58953-7

These days, the sterilization of medical instruments is a must in hospitals, laboratories and the medical industry. As such, proper handling of medical instruments begins even before sterilization by choosing the right packaging material and ensuring that the sterile package is perfectly sealed. This is the only way of guaranteeing instrument preparation that offers maximum hygiene and safety.

The hm 800 DC is the ideal sealer for medical packaging in hospitals, laboratories and the medical industry. It satisfies the highest quality and safety standards.

Microprocessor-controlled printing unit

Single-line printing above the sealed seam during the sealing process:

- Automatically updated sterilization and use-by date
- Batch and personnel number
- Symbols as per EN 980
- Text memory
- Variable font width
- Ribbon replacement from the front
- Internal printer can be switched off
- Programmable piece counter
- Elapsed hour meter

Functions

- Menu-driven device operation via digital LCD display
- Programmable in several languages: standard texts in German, English, French, Italian and Spanish
- Elapsed hour meter
- Clock and calendar function
- Batch counter can be reset
- Data storage and date updates even when device is switched off
- Motor start cut-out in the event of 5°C (41°F) deviation from the preset temperature value

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Technical data

Printing unit	1-line
Printing start margin	variable in mm (in.)
Sealing speed	10 m/min (32.8 ft./min)
Sealing margin	5 – 30 mm (0.2 – 1.2 in.)
infinately variable	
Sealing seam	12 mm (0.5 in.)
(hawoflex sealing system)	
Safety distance sealing	> 30 mm (> 1.2 in.)
seam to packed goods	
Temperature control	Microprocessor
Sealing temperature	80 – 220°C
	(176 – 428 °F)
Temperature tolerance	< ± 2 %
Output	500 Watt
Power supply	230 V / 50 Hz
	115 V / 60 Hz
Dimensions (w x d x h)	620 x 260 x 250 mm
	(24.4 x 10.2 x 9.8 in.)
Weight	23 kg (51 lb.)

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hawo GmbH · Obere Au 2-4 · 74847 Obrigheim / Germany